


Pad Cutting Pattern:

6 mm Wide EVA Pads (Firm & Soft) – Sizes 85W–145W

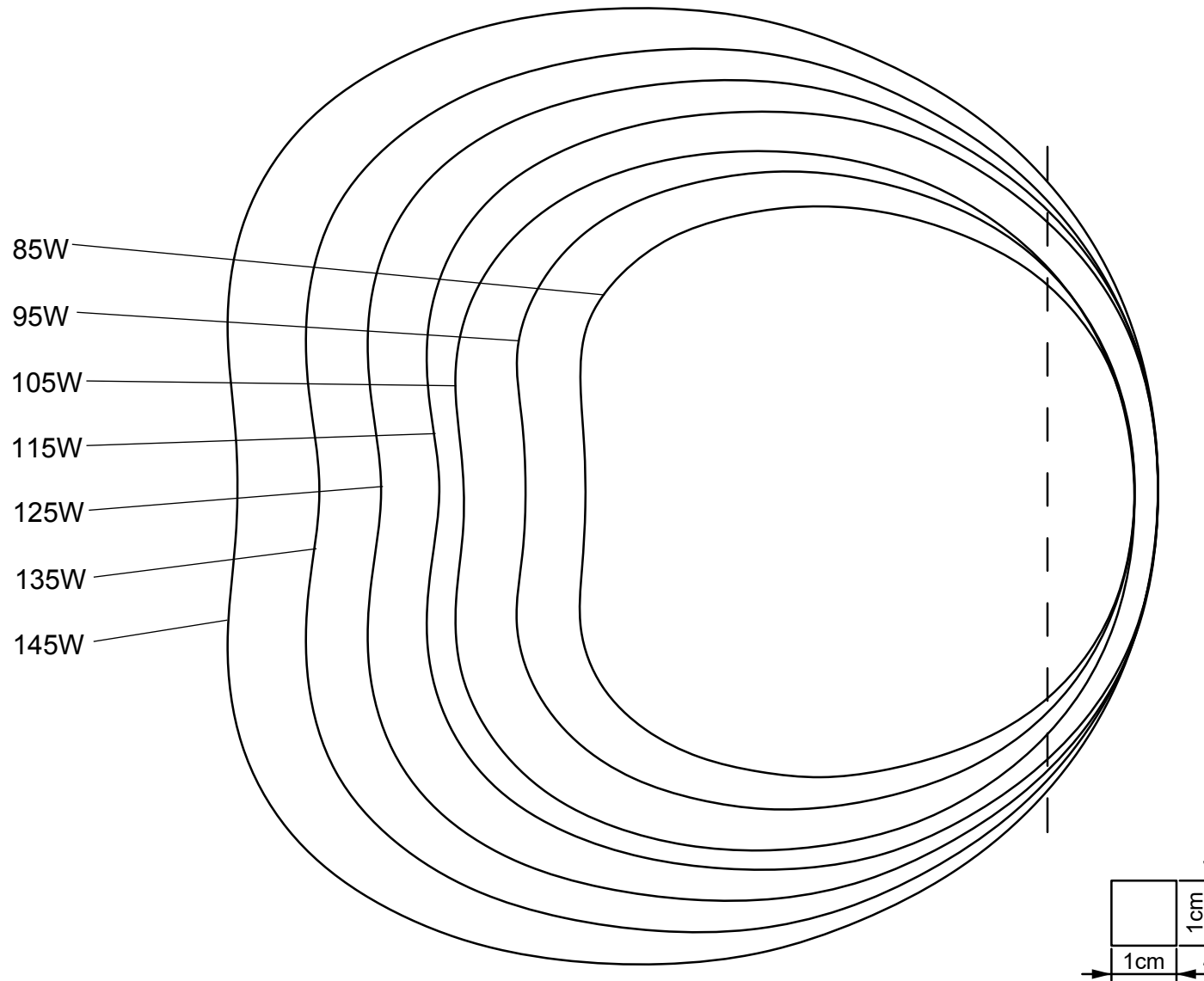

Flex Hoof Boots



Flex Pad Instructions

6 mm Pads



Flex EVA Pad - Firm & Soft:

Cut the pads to size following the outline. When fitting the boot with the pad inside, make sure the hoof's toe reaches **fully into the toe area** of the boot. You can gently tap the toe into place using a rubber mallet or similar tool.

During use, the EVA pad may expand slightly beyond the edge of the hoof. This is not an issue - on the sides, it can even **improve the fit** by filling small gaps and adding stability. However, **extra material at the toe area** can prevent the hoof from reaching the boot's toe section properly, which may interfere with the natural breakover.

Flex Boots feature the **most anatomically correct breakover point on the market**, and we want to ensure your horse's natural biomechanics remain fully supported.

Care:

You can wash the EVA pads with lukewarm water, and extra plumpness can be obtained by drying them in a warm spot, such as in the sun/near the heater, or best of all – in a tumble dryer. Make sure not to leave them in a dryer for too long.